

## OPX mobilis® Operating tables Positioning examples





## A Sense of Company

With more than 90 years of experience, SCHMITZ is one of the leading international manufacturers of medical products.

A family-owned business with a living tradition. Our typical employee has been with the company for some 18 years, proof of how strongly they identify with our values and philosophy.

In 2012, the company's management passed to Friedrich and Matthias Schmitz, the fourth generation of the family.

We are proud to be located in Germany and are deeply rooted in our home region. We develop and manufacture here.

90% of our suppliers are from Germany, of which 60% are from the local region.

## Made in Germany





## **A Sense of Care**

We understand the services which are our lifeblood, and aim to achieve maximum customer satisfaction. From consulting through planning to installation.


Cost-effectiveness through longevity is what you can expect from us.

With a well thought-out service concept and comprehensive support, SCHMITZ works alongside you as partner from start to finish.

### **Technical Customer Service**

T +49 (0)2377 84 549

[service@schmitz-soehne.de](mailto:service@schmitz-soehne.de)

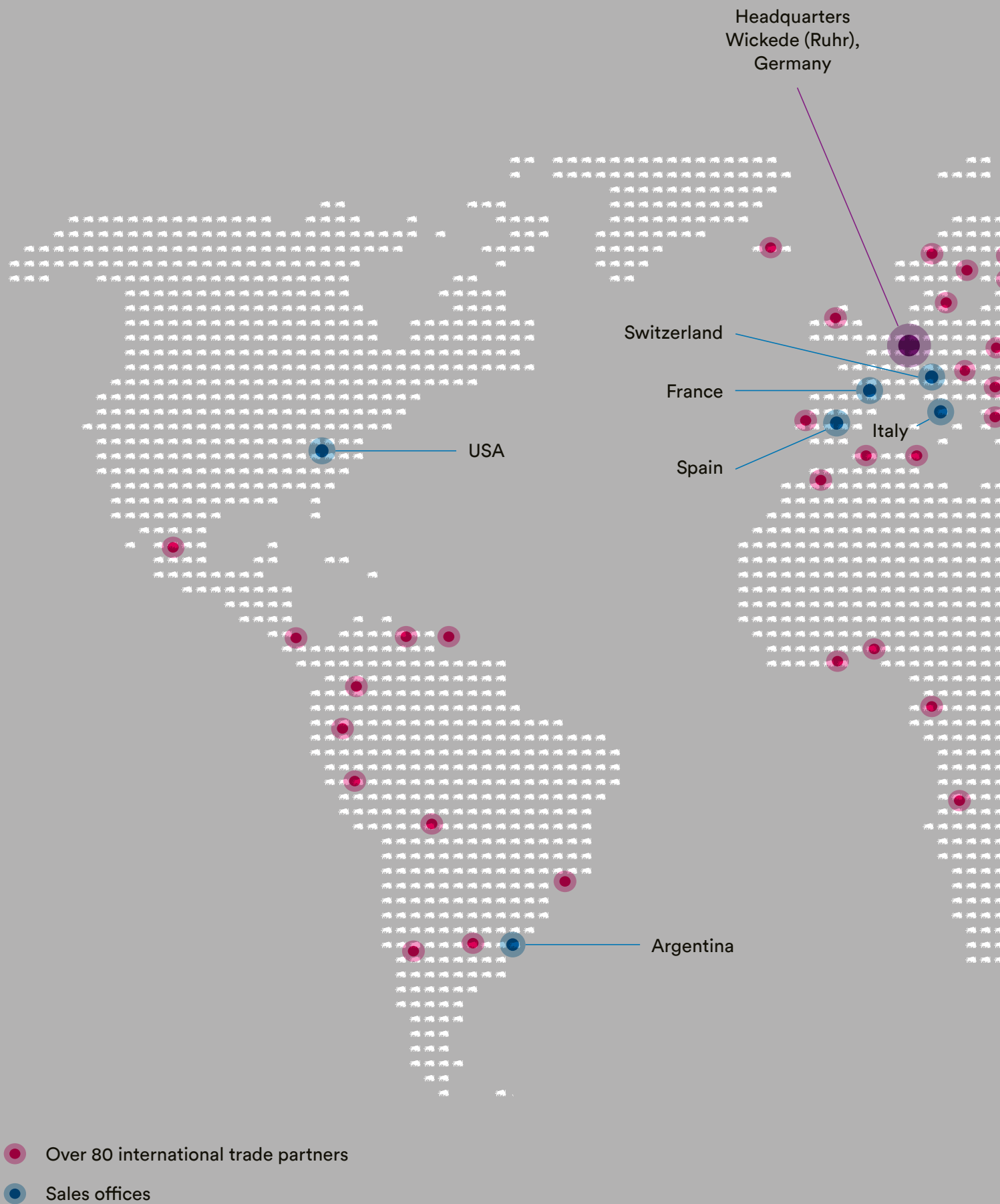

## SCHMITZ – represented worldwide





**OPX mobilis® CE<sup>MD</sup>****The all-rounder of operating tables****Functional operability**

The straightforward, reliable operating concept allows simple and independent positioning.

Universal, flexible and versatile.

**Comfortable and preventive**

The use of memory foam pads guarantees maximum patient comfort and ideal decubitus prophylaxis.

**Universal mobility**

The stable chassis ensures 100% controlled and energy-efficient mobility for added safety.

**Easy to care for and hygienic**

The sophisticated holistic hygiene concept guarantees optimal hygienic conditions in the OR theatre thanks to self-adhesive gel glue.

**Universal, flexible and versatile**

The manual or electrohydraulic adjustment options together with an extensive range of accessories for the mobilis make this a universal and versatile operating table – always adapted to your needs and working methods.





Femoral shaft fractures



Fractures of neck of femur



Lower leg fractures





---

Shoulder surgery





Kidney position



Urology



Neurosurgery in supine position



Neurosurgery in Beach-chair position



ENT, OMFS, Ophthalmology



OMFS, Ophthalmology



## Contact

SCHMITZ u. Söhne  
GmbH & Co. KG  
Zum Ostfeld 29  
58739 Wickede (Ruhr)

P. O. Box 14 61  
58734 Wickede (Ruhr)  
Germany

T +49 (0)2377 84 0  
F +49 (0)2377 84 162  
zentrale@schmitz-soehne.de

Exports department:  
T +49 (0)2377 84 0  
F +49 (0)2377 84 135  
export@schmitz-soehne.de

Technical service, hotline:  
T +49 (0)2377 84 549  
F +49 (0)2377 84 210  
service@schmitz-soehne.de

Information on our  
establishments worldwide  
can be found  
on our website:

[www.schmitz-soehne.com](http://www.schmitz-soehne.com)



All devices marked with the CE-MD symbol, in combination with our accessories, meet the requirements of Medical Device Regulation 2017/745/EU (MDR) as well as the Medical Device Amendment Act and are risk class 1 medical devices.



All devices marked with the simple CE symbol meet the requirements of low-voltage directive 2014/35/EU as well as the regulation on electrical equipment.

Please also refer to the instructions for use, for more details about the intended use!



Our goods and services are to be understood ex works. Deliveries are effected via our authorized partners or branch offices. The company reserves the right to alter the design, construction, dimensions and finish of its products without notice. All relevant dimensions and weights are to be understood as approximate information/data. The patient positionings shown in this catalogue are only exemplary and make no claim to the correctness and/or completeness of the accessories, please observe the instructions for use in any case. Due to technical reasons colour deviations are possible.  
Nominal charge 3.00 Euro.